

HEALTH HUB

Connects you with your care team

Plug it In

- » Plug in the Health Hub Console.

Step
01



Power On

- » Flip the Power Switch to the "ON" position (Located on the bottom of Console).
- » Your Health Hub will automatically connect to our cellular network.

Step
02



Press Call button

- » Press the lower left hand blue button to test the connection and make sure that it's properly working.

Step
03



PULSE OXIMETER

Measures blood oxygen and pulse rate

Clip sensor on the index finger

- » Clip sensor on your index finger, fingernail side up, as indicated. Keep your hand still.

Step
04



Press the "ON" button

- » Press the "ON" button on your Pulse Oximeter and wait 30 seconds.

Step
05



The results of your reading

- » Your Health Hub system will announce the results of your reading. Remove device.

Step
06



**YOU'RE READY
AND PROTECTED!**



www.nightcare.com